**3M**

# Polyether Adhesive

- (en)** Adhesive
- (de)** Adhäsiv
- (fr)** Adhésif
- (it)** Adesivo
- (es)** Adhesivo
- (pt)** Adesivo
- (nl)** Adhesief
- (el)** Συγκολλητικός παράγοντας
- (sv)** Adhäsiv
- (fi)** Sidosaine
- (da)** Adhæsiv
- (no)** Adhæsiv

Instructions for Use  
Gebrauchsinformation  
Mode d'emploi  
Informazioni per l'uso  
Información de uso  
Instruções de uso

Gebrauchsinformatie  
Οδηγίες χρήσης  
Bruksanvisning  
Käyttöinfo  
Brugsanvisning  
Bruksinformatjon

**3M Deutschland GmbH**  
Dental Products  
Carl-Schurz-Str. 1  
41453 Neuss - Germany

3M Customer Care/MSDS Information:  
U.S.A. 1-800-634-2249 and Canada 1-888-363-3685.

3M is a trademark of 3M or 3M Deutschland GmbH.  
Used under license in Canada. © 2015, 3M. All rights reserved.

4400076994/102

## (en) ENGLISH

### Product Description

Polyether Adhesive creates tight adhesion between impression trays and polyether impression materials from 3M.

These Instructions for Use should be kept for the duration of product use.

### Indications

- Impression trays made of metal, methacrylate, or polypropylene
- Polyether impression materials

### Precautionary Measures

Polyether Adhesive is highly flammable. Avoid static electricity build-up. Keep away from heat, sparks, open flame, pilot lights, and other ignition sources.

3M MSDSs can be obtained at [www.mmm.com](http://www.mmm.com) or from your local subsidiary.

### Application

- ▶ After testing the impression tray for fit in the patient's mouth, clean the tray to prevent any contamination of the adhesive brush.
- ▶ Use the brush to apply a thin layer of the adhesive to the tray.
- ▶ Allow to dry completely; 1–15 minutes are ideal.
- ▶ Carefully close the bottle tightly after use.
  - Before closing the bottle, remove any adhesive remnants from the threads of the bottle and screw cap with a cloth soaked in acetone or ethanol.

product for user's application. If this product is defective within the warranty period, your exclusive remedy and 3M Deutschland GmbH's sole obligation shall be repair or replacement of the 3M Deutschland GmbH product.

### Limitation of Liability

Except where prohibited by law, 3M Deutschland GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence, or strict liability.

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